

**HACKETTSTOWN REGIONAL MEDICAL CENTER
JOAN KNECHEL CANCER CENTER
WATER PRESSURE**

Effective Date: February 2013

Policy No: ROC TECH 92

Cross Referenced:

Origin: Radiation Oncology

Reviewed Date:

Authority: Executive Director

Revised Date:

Page: 1 of 1

SCOPE:

Department of Radiation Oncology

DEFINITIONS:

N/A

PURPOSE

To establish a process for monitoring water pressure in the treatment machine

POLICY

It is the policy of the Cancer Center to follow the procedure to monitor water pressure.

PROCEDURE

- 1) Linear accelerator water pressure shall be checked daily as part of the daily QA;
- 2) Varian 21EX water pressure range is 78-80 psi
- 3) If water pressure not within the range noted above, DO NOT use Linac – notify Oncology Services International (OSI) 800-445-4516.