

**HACKETTSTOWN REGIONAL MEDICAL CENTER
CARDIO PULMONARY POLICY MANUAL
DISPOSAL OF BLOOD WASTE CARTRIDGES**

=====

Effective Date: July 2009

Cross Referenced:

Reviewed Date: 10/15

Revised Date: 06/12

Policy No: 4A.017

Origin: Cardio Pulmonary

Authority: Cardio/Pulmonary Manager

Page: 1 of 1

Purpose: To properly dispose of blood waste cartridges from Blood Gas Analyzers.

Procedure:

At the analysis screen, touch the Status button.

At the Status screen, touch the Wash/Waste cartridge button.

NOTE: Ensure that nothing is blocking the door.

Touch Replace.

Open the door.

Remove the wash/waste cartridge and dispose of this cartridge in the red garbage bag.