

**HACKETTSTOWN REGIONAL MEDICAL CENTER
CARDIO PULMONARY POLICY MANUAL
SHUTTING DOWN AND RESTARTING THE SYSTEM**

=====

Effective Date: July 2009

Cross Referenced:

Reviewed Date: 10/15

Revised Date: 06/12

Policy No: 4A.018

Origin: Cardio Pulmonary

Authority: Cardio/Pulmonary Manager

Page: 1 of 1

Purpose: Use this procedure to shut down the system before you perform service and to restart the system when finished.

Procedure:

Touch the Status button.

Touch Shutdown.

When prompted, turn the power switch off.

To restore power to the system, turn the power switch on.

Cartridges remain stable for 60 minutes without power.