

**HACKETTSTOWN REGIONAL MEDICAL CENTER
CARDIO PULMONARY POLICY MANUAL
ANALYZER STANDBY PROCEDURE**

=====

Effective Date: June 2012
Cross Referenced:
Reviewed Date: 10/15
Revised Date: 06/12

Policy No: 4A.021
Origin: Cardio Pulmonary
Authority: Cardio/Pulmonary Manager
Page: 1 of 1

Purpose: Use this procedure to place one analyzer into standby when placing the other analyzer into use.

Procedure:

To place "ON" standby.
Press computer icon (top right hand corner of screen).
Press "system info" on bottom of screen.

To place "OFF" standby.
Press "continue" on system info screen.
Press Green Button to return to sample analysis screen.
Or if prompted to, change out cartridges.
See Appendix pages 10-11 for appropriate cartridge replacement instructions.