

Section: Division of Nursing

PROCEDURE

Index: #10
Page: 1 of 1
Issue Date: March 16, 1998
Reviewed Date: July 2010, 8/2011

HACKETTSTOWN REGIONAL MEDICAL CENTER

MONITOR TECH'S MANUAL
(Scope)

TITLE: CHANGE THE DATE OR TIME ON TELEMETRY MONITORS

PURPOSE: To change the date or time on telemetry monitors:

CONTENT: PROCEDURE STEPS KEY POINTS

1. Select Setup CIC from bottom right of either central station monitor.
 2. Select Service Password tab form new window in upper right.
 3. Type password: mms_com and a new window pops up with this prompt:
C:\Program Files\Marquette\CIC\4.0.8
 4. At this prompt type:
su cicadm cicadm1,3,5,7
 5. Another new window comes up with the same prompt as before:
C:\Program Files\Marquette\CIC\4.0.8
At this prompt type: timedate
- You will now have the window you need to change the time and date.