

Section: Division of Nursing

\*\*\*\*\*

Index: 7420.029a

**PROCEDURE**

Page: 1 of 1

Approval: \_\_\_\_\_

\*\*\*\*\*

Issue Date: August 29, 1995

Review Date: January 21, 2005

HACKETTSTOWN COMMUNITY HOSPITAL

Originator: Anita Beardsley, RN

Reviewed by: W. Zeiher, RN

**OR**  
(Scope)

**TITLE: OPERATION OF G.U. WOLF LIGHT SOURCE**

---

PURPOSE: Proper use and care of G.U. Wolf Light Source.

CONTENT: PROCEDURE STEPS:

KEY POINTS:

1. Move cart with Light Source in room.

Kept in Equipment room.

2. Plug in Light Source.

If bulb burned out, replace according to attached manual pages 3 and 4.

3. Plug light cord into source.

4. Adjust reostat to proper setting.

On completion of procedure, return reostat to 1.

5. Return light source and cart to Equipment room.

Reference: Richard Wolf Fiber Light Projection Manual; Richard Wolf Medical Instruments Corp, Rosemont, IL