

Section: Division of Nursing
Approval: _____

PROCEDURE

Index: 7420.031a
Page: 1 of 1
Issue Date: July 1, 1996
Review Date: Mach 12, 2008

HACKETTSTOWN REGIONAL MEDICAL CENTER

Originator: Carol Wicki, RN, CNOR
Judy Young, RN
Reviewed by: O. Lopez, RN, BSN, CNOR

OR/CSR
(Scope)


TITLE: STERIS: CHANGING PRE-FILTERS

PURPOSE: To outline the steps for changing the filters.

CONTENT: PROCEDURE STEPS:

1. Turn the water OFF.
2. **Exhaust Air:**
Press the red button located on the top of the pre-filter housing to exhaust air.
3. **Remove Housing:**
Unscrew counter-clockwise the housing body.
4. **Remove Filter:**
Rinse the housing bowl with clean water.
5. **Install New Filter:**
Filters are placed into the bowl with the **black "O" ring facing up.**
6. Turning clockwise, screw the filter bowl into the housing.
7. Turn the water back on.
8. **Exhaust Air:**
Press the red button located on top of the housing.

Changing Pre-Filters

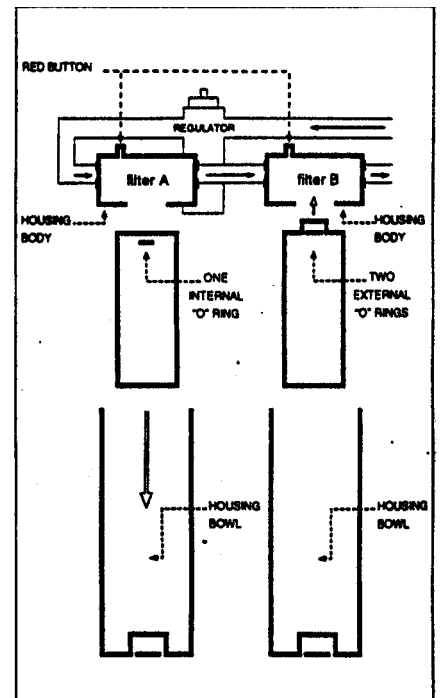

Reference: Steris System 1 Operation Manual (1994)