

Section: Division of Nursing
Approval: _____

PROCEDURE

Index: 7420.051a
Page: 1 of 1
Issue Date: March 23, 2005

HACKETTSTOWN REGIONAL MEDICAL CENTER

Originator: P. Stafford, RN
Reviewed by: W. Zeiher, RN

OR
(Scope)

TITLE: INSTRUCTIONS FOR HALL BURR GUARD

PURPOSE: To instruct the OR Nurse in the proper use of Hall Bur Guard.

- EQUIPMENT:
1. Bur Guard
 - a. Medium
 - b. Long
 - c. Extra Long

CONTENT:

PROCEDURE STEPS:

1. Slide the guard over the end of the drill so it is completely seated.

KEYPOINT

1. Select a bur guard the proper length of bur being used.

WARNINGS:

1. Never operate the Hall Surgitone II without appropriate bur guard. The bur guard can break without proper stabilization.

Reference: The Hall Micro 100 Instruction Manual