

Section: Division of Nursing

* **PROCEDURE** *

Index: 7420.057a
Page: 1 of 1
Issue Date: March 25, 2005

Approval: _____

HACKETTSTOWN REGIONAL MEDICAL CENTER

Originator: O. Lopez, RN BSN CNOR
Reviewed by: W. Zeiher, RN

OR
(Scope)

TITLE: HALL MICRO 100 RECIPROCATING SAW

PURPOSE: To outline the steps for proper use of the Micro 100 Reciprocating Saw.

EQUIPMENT:

1. Micro Reciprocating Saw
2. Pneumatic Cord
3. Appropriate Blade

PROCEDURE:

1. To insert saw blades:
 - a. Twist the blade lock to loosen.
 - b. Insert the blade into the end of the handpiece. Be certain the blade is fully seated.

2. To secure blade:
 - a. Twist the blade lock until it is completely tightened.
 - b. Active the instrument briefly and retighten the blade lock.

WARNING:

1. Be sure the instrument control lever is placed in the SAFE position before inserting blades.
2. Be certain saw blades are secured tightly before use.
3. **DO NOT** exceed 100 psi running.

Reference: Linvactec Hall Micro 100 Instruction Manual