

Section: Division of Nursing

\*\*\*\*\*

Index: 7420.062a

**PROCEDURE**

Page: 1 of 1

Approval: \_\_\_\_\_

\*\*\*\*\*

Issue Date: May 29, 2002

Reviewed Date: March 12, 2010

HACKETTSTOWN REGIONAL MEDICAL CENTER

Originator: M. Mann, RN, CNOR

Reviewed by: R. Trimmer RN

**OR**  
(Scope)

**TITLE: SET UP AND USE OF THE SENO RX GAMMA FINDER**

---

**PURPOSE:** To outline the set-up and use of the Seno Gamma Finder.

- PROCEDURE:**
1. Obtain Gamma Finder (Seno Rx) from protective case.
  2. Obtain sterile sleeve and have sterile scrub remove from pouch.
  3. Have sterile scrub hold open sterile pouch and place Gamma finder (Seno Rx) into sterile pouch.
  4. To switch on Gamma finder (Seno Rx):
    - a. Press any button
    - b. Press key **D** to access configuration menu. To increase valve press key **B (+)** to decrease valve press key **C (-)**.
    - c. Press **(:)** key once to enter volume, press **(+)** key to increase and **(-)** to decrease volume.
    - d. Press light key once to turn on light
    - e. To adjust range (1x) press **(I)** key to adjust and **(+)** key to increase and **(-)** to decrease.
    - f. At the end of case, remove soiled sterile sleeve and discard.
  5. To clean the Gamma finder (Seno Rx), use a moist cloth soaked in solvent free detergent of disinfectant **DO NOT SOAK**.
  6. After five (5) minutes the Gamma finder (Seno Rx) will turn off, place in metal case and return for safe storage.