

Section: Division of Nursing

\*\*\*\*\*

Index: 7420.007a

**PROCEDURE**

Page: 1 of 1

Approval: \_\_\_\_\_

\*\*\*\*\*

Issue Date: March 16, 2005

\_\_\_\_\_

HACKETTSTOWN REGIONAL MEDICAL CENTER

Originator: O. Lopez, RN, BSNCNOR

Reviewed by: W. Zeiher, RN

**OR**  
(Scope)

**TITLE: HALL MICRO 100 OSCILLATING SAW**

---

**PURPOSE:** To outline procedure for proper use of Micro 100 Oscillating Saw.

A. To attach blade

1. Place the blade between the end of the output shaft and the compression washer

**WARNING:** Be sure the instrument control lever is placed in the "**SAFE**" position before inserting blades.

**NOTE:** Do not place the blade between the washer and the head of the screw.

B. To secure blade:

1. Tighten the lock screw with the hex wrench (1365-005).
2. Activate the instrument briefly and retighten the lock screw.

**WARNING:**

1. DO NOT exceed 100psi running.
2. Be certain saw blades are tightly secured before use.
3. The blades may be positioned at any angle.