

Section: Division of Nursing

* **PROCEDURE** *

Index: 7420.012a
Page: 1 of 1
Issue Date: March 16, 2005

Approval: _____

HACKETTSTOWN REGIONAL MEDICAL CENTER

Originator: P. Stafford, RN
Reviewed by: W. Zeiher, RN

OR
(Scope)

TITLE: HALL MICRO 100 SAGITAL SAW

PURPOSE: To instruct the OR nurse in the proper use of the Micro 100 Sagital Saw.

- EQUIPMENT:**
1. Micro 100 Sagital Saw
 2. Pneumatic Cord
 3. Appropriate Blade
 4. Hex Wrench

CONTENT:

PROCEDURE STEPS:

1. Attach saw blade
 - a. Place blade in the center slot at the end of the handpiece.
2. Secure blade:
 - a. Tighten the lock screw with the hex wrench.
 - b. Activate the instrument briefly, then retighten the lock screw.

KEY POINTS:

The blade may be positioned anywhere within a 180° arc..

Be sure blades are tightly secured before use
For maximum efficiency handpiece should run at 100 psi.

WARNING:

1. Be sure the instrument control lever is placed in the safe position before inserting blades.
2. DO NOT exceed 100 psi running.

Reference: The Hall Micro 100 Instruction Manual