

Section: Division of Nursing

\*\*\*\*\*  
\* **PROCEDURE** \*  
\*\*\*\*\*

Index: 7420.023a  
Page: 1 of 1  
Issue Date: March 21, 2005

Approval: \_\_\_\_\_  
\_\_\_\_\_

HACKETTSTOWN REGIONAL MEDICAL CENTER

Originator: P. Stafford, RN  
Reviewed by: W. Zeiher, RN

**OR**  
(Scope)

**TITLE: SERIES 4 OSCILLAR**

---

**PURPOSE:** Instruct OR personnel in the proper use of the Series 4 Oscillar.

**EQUIPMENT:**

1. Series 4 Oscillator
2. Power Cord

**CONTENT:**

**PROCEDURE STEPS:**

**KEY POINTS:**

1. To affix blade to blade – locking – collet, open blade locking mechanism by rotating the blade locking knob 90° to open position.
2. Insert blade at desired angle and align blade holes with the blade positioning pins.
3. Rotate blade locking knob to closed position.
4. The head position collar may be set in any of 12 positions at 30° intervals.
5. To change position and cutting planes:
  - a. Place safety knob in safe position.
  - b. Grasp the head position collar firmly and pull toward the instrument.
  - c. Rotate the head and blade to desired position and release the head positioning collar.

Make sure handpiece is in safe position.

Use only the proper saw blades with the collet.

Operate at 100 psi running.

**WARNING:** Instrument should always be in safe position when changing hoses, saw blades or not in use.

Reference: The Hall Series 4 Instruction Manual.