

Section: Division of Nursing
Approval: _____

PROCEDURE

Index: 7420.033a
Page: 1 of 1
Issue Date: July 1, 1996
Review Date: March 10, 2008

HACKETTSTOWN REGIONAL MEDICAL CENTER

Originator: Carol Wicki, RN, CNOR
Judy Young, RN
Reviewed by: O. Lopez, RN, BSN, CNOR

OR/CSR
(Scope)


TITLE: PROCEDURE TO CHANGE THE STERIS AIR FILTER

PURPOSE: To outline the steps to change the air filter.


CONTENT: PROCEDURE STEPS:

KEY POINTS:


1. Open the control panel and locate the sterile air Filter.
2. Remove the air filter. Turn the air filter counter Clockwise, lift up and remove.

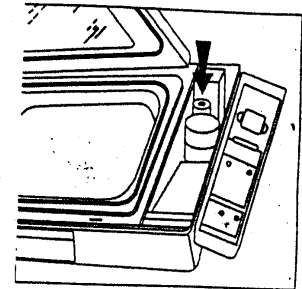

3. Remove air filter. Turn the air filter counter clockwise, lift up and remove.

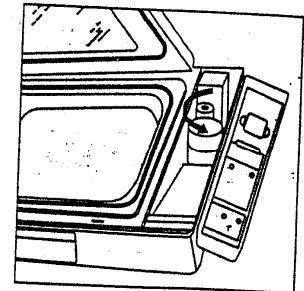

4. Replace with the new air filter, with **flow arrow pointing down, turn filter clockwise.**

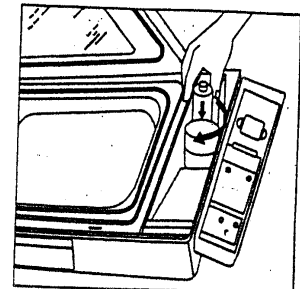

5. Close the control panel

SPECIAL/SAFETY POINTS:

1. Be sure the power is off.
2. The filter is to be changed every six months