

Section: Division of Nursing
Approval: _____

PROCEDURE

Index: 7420.054a
Page: 1 of 1
Issue Date: March 23, 2005

HACKETTSTOWN REGIONAL MEDICAL CENTER

Originator: P. Stafford, RN
Reviewed by: W. Zeiher, RN

OR
(Scope)

TITLE: HALL WIREDRIVER 100

PURPOSE: To instruct OR nurse in the proper use of the Hall Wiredriver 100.

EQUIPMENT LIST:

1. Hall Wiredriver 100
2. Pneumatic Cord
3. Appropriate Wire
4. Wire Guard Protector

CONTENT:

PROCEDURE STEPS:

KEY POINTS:

1. To insert wires:
 - a. Be certain the directional control lever is in safe position.
 - b. Wires may be inserted either through the front or the rear of the handpiece.
 - c. Wire guard protector may be attached to rear of instrument to keep long wires from bending.
 2. To lock the wire in a fixed position for use:
 - a. Depress the trigger to lock the collet on the wire.
- a. Inserting wires at an angle may cause them to become lodged.
 - b. Only wires 6" or longer can be inserted from the rear.
 - c. When inserting wires from the front, care should be taken to align the wire with the front of the instrument.
 - d. The instrument must be in reverse position to remove threaded wires.

WARNINGS:

1. Be sure instrument control lever is in safe position before inserting wires.
2. **DO NOT** exceed 100 psi running.

Reference: The Hall Micro 100 Instruction Manual.